President Dental

catalogue
katalog

Universal Hybrid Composite Restorative
Dynamic Plus PROCEM *ProShield* ETCHING GEL PROCRYLA DUROSIL L
NOVA-CAPS *varnish* NOWAX **NOVAMIX** NOVALLOY
FORMALLOY H PASTE HARDENER NOVA 70-CAPS **PREBOND** (EMEL-U) **NOVA-TRAY**
NOVASEAL CERAPLUS S **ENDOPLUS** (EMEL-I) PROPHY POWDER Absorbent Paper Points
FORMALLOY S DUROSIL S **ZENIT** PREBOND SE Gutta Percha Points
ENDOPEX Light Curing Nano Hybrid Composite CERAVEST *Dynamic Flow* **NOVA 70**

President Dental

katalog
catalogue

■ Made
■ in
■ Germany

-best quality
-high standards

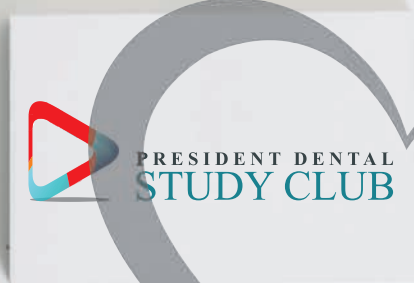
**The
revolution
in dentistry**

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www.presidentdental.de

PRESIDENT DENTAL STUDY CLUB

“Follow President Dental Product Clinical Cases On

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**NATIONAL EVALUATION CENTER
OF QUALITY & TECHNOLOGY
IN HEALTH S.A.**

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- Morocco
- Libya
- Egypt
- Algeria
- Sudan
- Ghana
- Mauritania

- Germany
- Poland
- France
- England
- Spain
- Italy
- Lithuanian

- Kosova
- Greece
- Moldova
- Bosnia&Herzegovina
- Bulgaria
- Czech Republic
- Albania
- Macedonia
- Romania
- Serbia
- Hungary

- United Arab Emirates
- Kuwait
- Bahrain
- Jordan
- Iran
- Iraq
- Turkey
- Israil
- Lebanon
- Yemen
- Saudi Arabia
- Qatar

- Russia
- Azerbaijan
- Georgia
- Kazakhstan
- Turkmenistan
- India
- South Korea
- Malaysia
- Indonesia
- Philippines

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ZENIT
LC Nano Ceramic Composite

Dynamic Plus
Universal Hybrid Composite Restorative

Zenit BRUSH
pure kolinsky dental composite brush

ZENITcolor
Light - Curing Stains On Composite Basis

Dynamic Flow
Universal Flowable Restorative Composite

zenitcem
self adhesive resin cement

• Bonding & Etching Materials / *Bonding & Etching Materialien*

Prebond	20
Prebond SE	21
Prebond Dual	22
Etching Gel	23

PREBOND
5th Generation Adhesive System

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PROCEM
Free - Luting Cement

extracem[®]

NOVASEAL
Light Curing Lining Material

ZENITflex
FINISHING AND POLISHING

• Endodontic Materials / *Endodontische Materialien*

Endoplus	28
Gutta Percha Point	29
Paper Points	30

ENDOPLUS
Root Canal Sealer

Absorbent Paper Points
Gutta Percha Points

• Preventive Products / *Präventive Produkte*

Proshield Varnish	32
-------------------	----

PROPHY POWDER
Prophylaxis Powder

• Prophylaxis / *Prophylaxe*

Prophy Powder	33
---------------	----

ProShield
5% sodium fluoride Varnish

• Impression Material & Accessories / *Abformmaterial & Zubehör*

Presigum	34
Durosil L	36
Durosil S	37
Paste Hardener	38
Durosil Kit	38
Cemel "U" Form	39
Cemel "I" Form	39
Sterilization Paper	40
Dental Mask	40
Dental Cotton	40
Alginate Mixer	41
Novatray	42

Presigum
Hydrophilic Addition Silicone / A- Silicone

NOVA-TRAY
Light - Curing Tray

DUROSIL
C-Silicone Impression Material

CEMEL-I CEMEL-U
Articulating Paper

NOVAMIX
Alginate Mixer

LABORATORY PRODUCTS / LABORPRODUKTE

• Zircon Block / *Zirkonoxid Block*

Zircon.X	43
----------	----

PROCRYLA
Acrylic Resin

• Acrylics / *Acryl*

ProCryla	44
----------	----

ZIRCONX
Advanced Ceramics for Crowns and Bridges
Super Translucency
High Mechanical Strength

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Formalloy H	46
Formalloy S	47
Ceraplust S	48

FORMALLOY S **FORMALLOY H** **CERAPLUST S**
Cobalt Chrome Casting Alloy Cobalt Chrome Casting Alloy Nickel Chrome Alloy for Ceramics

• Aluminum Oxide / *Aluminiumoxid*

Synthetic Abrasive	49
--------------------	----

Synthetic Abrasive

• Modelling Material / *Modelliermasse*

Nowax	49
-------	----

NOWAX
Modelling Wax

• Investment Material / *Einbettmasse*

Ceravest	50
----------	----

CERAVEST
Crown Bridge Investment

LIGHTING
POWER
MAX
2,700MW

ZENOLITE

LED CURING LIGHT

5 DIFFERENT OPERATION MODES

1 Standart mode

- 1) Optical output - Over 900 mW/cm² (±10%)
- 2) Available time setting – 5, 10, 15, 20 seconds

2 High mode

- 1) Optical output - Over 1,300 mW/cm² (±10%)
- 2) Available time setting – 3, 6, 9, 12 seconds

3 Soft start mode

- 1) Optical output - After gradually increasing from 0 to 1,300 mW/cm² (±10%) and be maintained until finish time
- 2) Available time setting – 3, 6, 9, 12 seconds

4 Orthodontic mode

- 1) Optical output - Over 1800 mW/cm² (±10%)
- 2) Available time setting – 3, 4, 5 seconds

5 Xtra mode

- 1) Optical output - Over 2700 mW/cm² (±10%)
- 2) Available time setting – 1, 2, 3 seconds

Zenit

Nano Ceramic Composite Kit
Nano-Keramik Komposit Kit

www.presidentdental.de

Product Code / Produktcode

PRD.01.100.ZN00

Package Details

- 7x 4gr Syringe (A1, A2, A3, A3.5, B2, A20, W2)
- 1x 5ml Prebond (5th Generation Adhesive System)
- 1x 3ml Etching Gel
- 1x 2gr Dynamic Flow Refill A2 shade
- 1x 50 pcs applicator
- 2x 2 pcs mixing well

EN

Verpackungsdetails

- 7x 4gr Spritzen (A1, A2, A3, A3.5, B2, A20, W2)
- 1x 5ml Prebond (5th Generation Adhesive System)
- 1x 3ml Etching Gel
- 1x 2gr Dynamic Flow Refill A2 shade
- 1x 50 stck Applikator
- 2x 2 stck Anmischblock

DE

Zenit Nano Ceramic Composite is a light-curing, highlustrer-polish hybrid- composite with an ultrafine, radiopaque porcelain filler for use in adhesive filling treatment. It can be polished to a highlustrer. Due to the ultrafine particle filler, extremely homogeneous restorations can be placed which are easily polished to a high lustre. The chameleon effect matches the shade of the filling perfectly to the tooth structure. The guidelines in **DIN EN ISO 24049** have been complied with. Zenit Nano Ceramic Composite is supplied in practical screw syringes.

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DE

Composition

Monomer Matrix: Diurethane dimethacrylate, butanediol dimethacrylate, isopropylidene-bis [2(3)-hydroxy-3(2)-(4-phenoxy) propyl] bismethacrylate.

Total filler content: 83% by weight (70% by volume)

- ✓ Glass filler (medium grit size 0.7 microns)
- ✓ Pyrogenic silica (medium grit size 12 nm)
- ✓ Agglomerated nanoparticles (medium grit size 0.6 microns)

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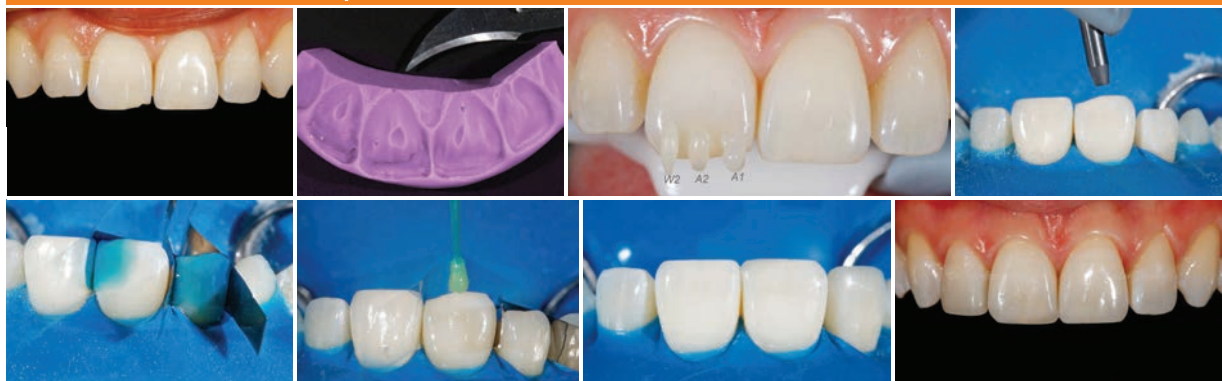
Indications

- ✓ Direct anterior and posterior restorations in all cavity classes
- ✓ Indirect restorations such as inlays, onlays, veneers
- ✓ Extended fissure sealing in molars and premolars
- ✓ Cores
- ✓ Adjusting contours and shade to improve aesthetics

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PRD.01.ZC.100

Package Details

- 10x 1gr Syringe Refill
Blue, Orange, Clear, Khaki, Dark Brown, Yellow, White, Light Brown, Black, Red

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DE

Zenit Color provides perfect characterization of composite facings on crowns and bridges, individualization of tailor-made resin teeth as well as characterization of denture base resins in partial and full dentures. Available in ten different shades.

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DE

Polymerization

Dialog Chroma-Flow can be polymerized in all conventional light-curing systems. Before the final polymerization, the complete polymerization site may be covered with expers-Gel. This avoids the formation of a new dispersion layer and facilitates the finalization of the object. Use a light-curing system with an emission spectrum of 310-500 nm. Perfect physical properties can only be obtained with immaculate lamps. It is important to test the light intensity regularly, in line with the manufacturer's instructions.

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Contra-Indications

If a patient has known allergies against or hypersensitivities towards a component of this product, we recommend not to use it or to do so only under strict medical supervision. The dentist should consider known interactions and cross reactions of the product with other materials already in the patient's mouth before using the product.

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DE

Composition Filler, Anorganic

(50 weight % respectively 36% by volume)
Glass filler (medium particle size: 0.7 µm)
Pyrogenic silica (medium particle size: 0.01 µm)

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DE

ZENIT color
Light - Curing Stains On Composite Basis

Zenit Flex Snap On

Finishing and Polishing Disc Kit
Finishing- und Polierscheiben Kit

www.presidentdental.de

Zenit Flex Snap On & Pop On Discs are ideal for high quality polishing of the materials such as composite, amalgam, glass ionomer, semi precious and precious materials.

- ✓ The design of the mandrel prevents sliding.
- ✓ Owing to the covered mandrel, the unwanted coloring is prevented
- ✓ Super thin folio disc can easily reach the desired area of the tooth and ideal for the inter-teeth polishing.
- ✓ The single side of the discs is covered with aluminium oxide.
- ✓ Easy to use because of color coding

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- 1x 50 Scheiben
- 1 Stk. Pop-On Mandrel

DE

ZENIT flex
FINISHING AND POLISHING

Zenit Flex Strips

Finishing and Polishing Stripes (Strip-Fine/Ultra Fine)

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www.presidentdental.de

Zenit Flex Stripe abrasive and polishing stripe is used for finishing and polishing the surface of the restoration. It is made of Aluminium oxide layer and polyester. Easy to snatch.

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Zenit Flex Strips

Finishing and Polishing Stripes (Strip-Coarse/Medium)

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Zenit Flex Stripes prevent the generation of interteeth plaques and assists to obtain high quality finishing and polishing process.

- ✓ The single side of the stripe is covered with aluminium oxide
- ✓ Super thin disc can easily reach the desired area of the tooth and ideal for the inter-teeth polishing
- ✓ Easy to use because of color coding

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EN

DE

Zenit Brush which is produced from pure Kolinsky enables you to make perfect composite retouches. Thanks to its ergonomic wooden handle, high level of clutch is provided and also enables you to make composite retouches with minimum effort and maximum quality.

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• There exists 3 different models of brusher for your different operations.

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www.presidentdental.de

DYNAMIC PLUS is a visible light cure microhybrid composite for anterior and posterior restorations. **DYNAMIC PLUS** is based on **BIS-GMA-Resin** and **Inorganic** filler particles of 0.05-1µm. The total filler load is 80%, and the total filler volume is 65%. It is available as single syringes or as kits in various combinations containing a bonding system and application accessories and various number of syringes. (5 syringes **Starter Kit** and 8 syringe **Standard Kit**).

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Composition

- ✓ Barium aluminosilicate (Average Particle Size: <1µm)
- ✓ Fumed silica (Average Particle Size: 0.04µm)
- ✓ Bis-GMA
- ✓ Triethyleneglycol dimethacrylate
- ✓ Others
- ✓ The percentage by weight of total inorganic filler is ca.80%

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- ✓ Easy handling, non-sticky formula that won't slump.
- ✓ Excellent wear resistance and superior performance at the margin.
- ✓ Resists chipping and fracturing for long-lasting restorations.

- Creamy consistency
- Great optical properties
- Simple technique
- Excellent polishability
- Long working time
- Suitable for all cavities in Anterior & Posterior teeth

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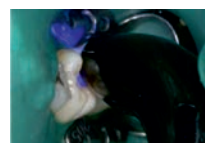
5. Light Curing /
Lichthärtung

6. Finishing /
Finishing

Dynamic Plus
 Universal Hybrid Composite Restorative

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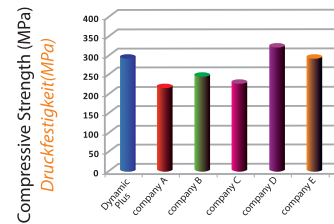
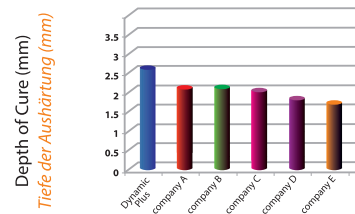
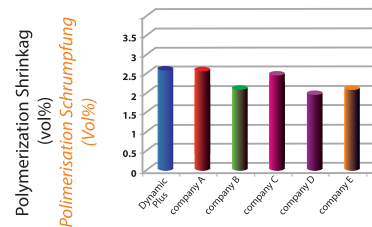
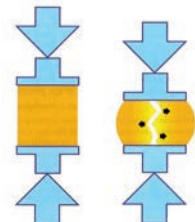
The ability of a material to resist fracture / breakage from compressive (or crushing) forces from a single impact

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Dynamic Flow

Universal Flowable Restorative Composite Set

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Verpackungsdetails

- 5x 2gr Drehspritzen (A1, A2, A3.5, B2, B3)
- 20x Applikatortips

Dynamic Flow is a flowable and restorative composite that can be polymerized with light and has low viscosity. Dynamic Flow that can provide perfect aesthetic with polishing retention presents you the universal technology. It provides excellent polishing and also less light cured.

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Key Features

- ✓ Excellent biocompatibility with dentin and pulp
- ✓ Light curing typer cavity liner, cure extremely hard
- ✓ Chemically bonds to adhesives and composites
- ✓ Contains calcium hydroxy phosphaste
- ✓ Releases calcium, hydroxyl, phosphate and fluoride ions
- ✓ Fluoride releasing
- ✓ Insoluable in water and oral fluids
- ✓ Radiopacity
- ✓ Flow on demand handling and easy dispensing

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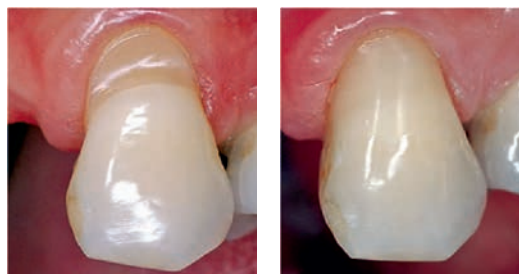
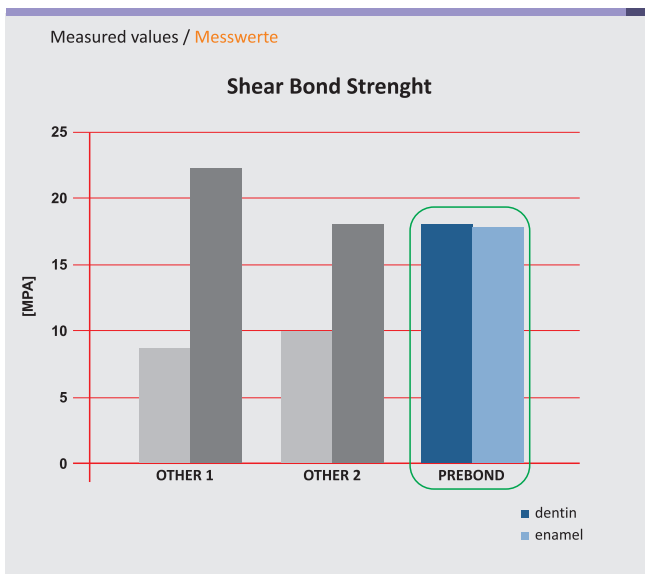
Package Details / Verpackungsdetails

- 1x 5ml

Prebond is a one component dental universal adhesive for bonding dentin/ enamel using adhesive restoration technique. It has an excellent adhesion value with a very high hydrophilic properties. The fluid is an easy flowing with excellent deep absorption into the retention thus enabling very high bond strenght while minimizing marginal gaps. Prebond Light cure universal bond 5ml.

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DE

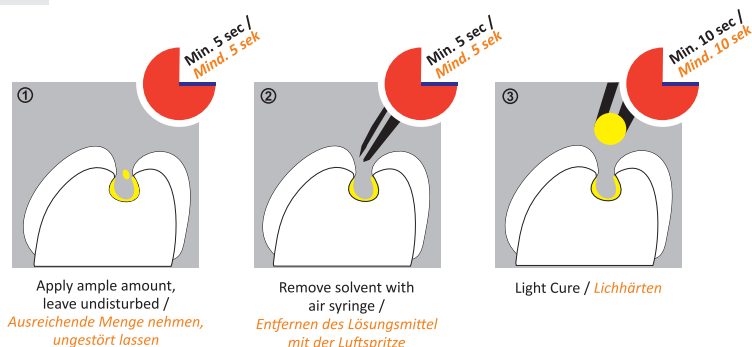
Prebond is used to improve the adhesion of composites on enamel and dentin. Further, it improves the adhesion between glass-ionomer underfill (lining) cement and composite fillings (sandwich technique)

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Prebond SE is a self-etching, light curing one component adhesive for the adhesive restorations technique used on direct and indirect restorations. Prebond SE is formulated to contain functional MDP monomers, methacrylate, photoinitiators, ethanol and water.

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Can be used on the following cases: Fixing of composite-material and compomer at enamel and dentin Adhesive fixing of restorations of ceramic or composite at enamel and dentin. Fixing and repairing of light cure composites at hybrid ceramic, composite, metal or amalgam Usable as dentin-sealer PREBOND SE Light cure universal bond self-etch 5ml. PREBOND SE light cure universal bond self-etch 5ml

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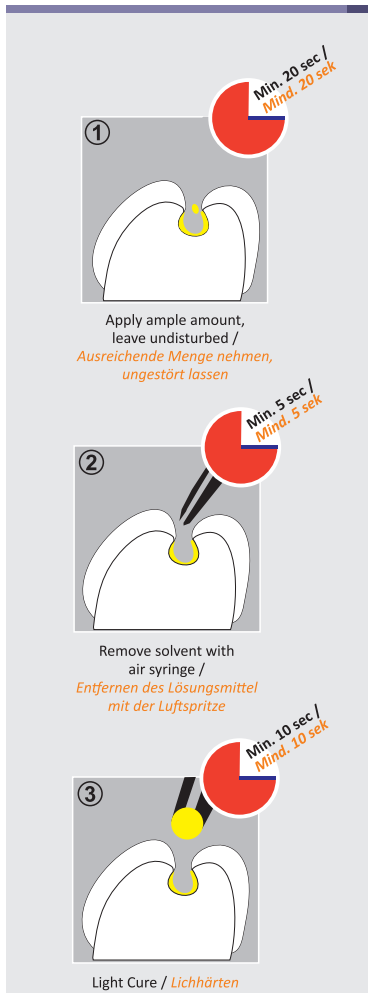
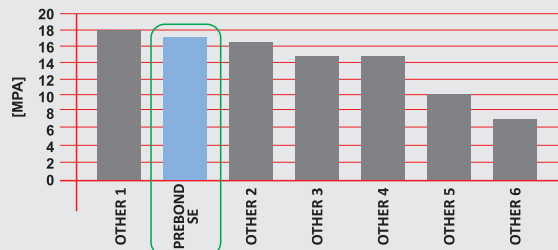
- 1x 5ml

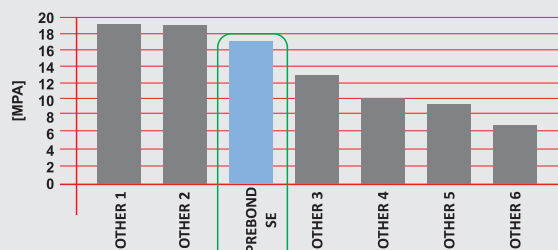
Measured values / Messwerte

Shear Bond Strength
Adhesion on bovine enamel, not etched

Measured values / Messwerte

Shear Bond Strength
Adhesion on bovine dentin, not etched

Light Cure
PREBOND SE
Universal bond self etch

Prebond Dual

Universal Adhesive Dual
Universal Adhesive Dual

www.presidentdental.de

Product Code / Produktcode

PRD.100.02.022

Package Details / Verpackungsdetails

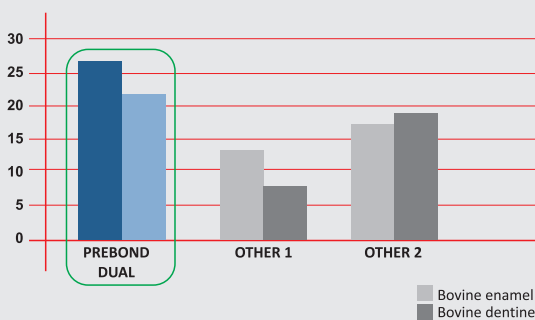
- 1x 5ml Bond A
- 1x 5ml Bond B

PREBOND DUAL

Universal Adhesive Dual

Measured values / Messwerte

Shear Bond Strength
on etched enamel and dentin
(chemical curing only)

Technical Data

Optional: Response time of the etch-ing gel to enamel	20 to 60 s
Optional: Response time of the etch-ing gel to dentine	Max. 15 s
Working time*	Max. 60 s
Mixing time	5 s
Working-in time	
As adhesive on tooth: 2 layers	Massage in for 2 x 20 s + blow for 10s
As primer on restorations: 1 layer	Massage in for 1 x 20 s + blow for 10 s
Optional: Light-curing time	At least 20 s

* When kept away from light during storage and mixing time, depending on the ambient temperature

Instructions for use "Universal adhesive dual"

Universal adhesive dual is a dual-curing two-bottle adhesion primer. The material can be used as an adhesive on enamel and dentine with the self-etch, total-etch or selective-etch technique in combination with dual-curing, self-curing or light-curing composites. Thanks to its dual-curing property, the material is particularly suitable for use in the root canal area where it is difficult for light to penetrate. The material can also be used as a primer for indirect restorations made of metal (base metal, precious metal), oxide and silicate ceramics or composites.

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Indications

As adhesive on enamel and dentine for:

- ✓ Direct fillings with light-curing composites and compomers
- ✓ Light-curing fissure-sealing materials
- ✓ Light-curing, self-curing and dual-curing core build-up materials
- ✓ Cementing of indirect restorations (inlays, onlays, crowns and bridges) and root posts with light-curing, self-curing and dual-curing composite-based luting cements
- ✓ Cementing of veneers with light-curing luting composites in conjunction with the enamel etching technique

As primer for indirect restorations:

- ✓ Metal (base metal, precious metal)
- ✓ Oxide ceramic (zircon oxide and aluminum oxide)
- ✓ Silicate ceramic
- ✓ Composites

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President Dental **Etching Gel** creates micropores in the enamel surface and removes the smear layer to unblock dentinal tubules. It promotes adhesive resin penetration and resin tag formation. President Dental **Etching Gel** is vital for the restoration's mechanical retention.

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DE

Properties

- ✓ **Etching Gel** is made up of 37% phosphoric acid gel
- ✓ Syringe offers ease of application for dispensing
- ✓ Thixotropic acid etching agent
- ✓ Blue color resulting visibility
- ✓ Good viscosity

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DE

Can be used in conjunction with all

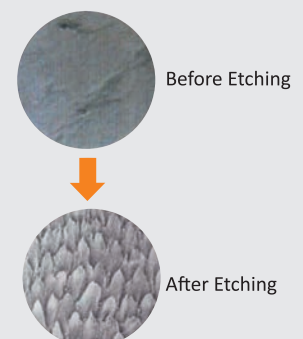
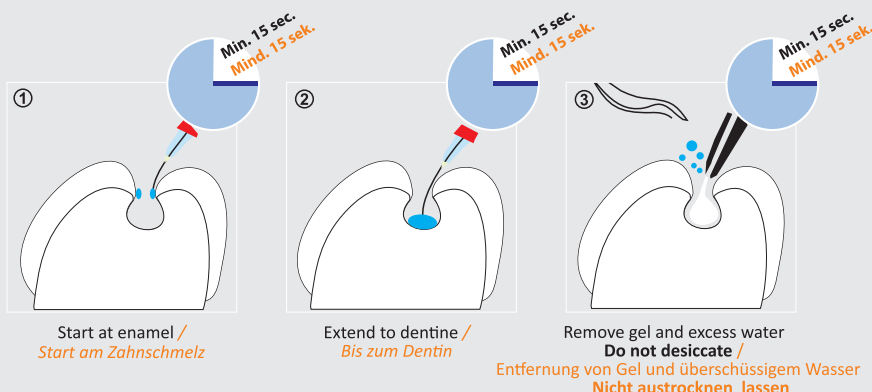
- ✓ Adhesives
- ✓ Composites
- ✓ Compomers
- ✓ Glass Ionomers
- ✓ Sealants

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- 1x 17ml Liquid
- 1x 35gr Powder

APPLICATIONS

Powder: Liquid Ratio	1 Scoop : 2 Drops / 2.00g : 1.00g
Mixing Time	30 seconds
Working Time (at 23 ± 1°C)	3 minutes 30 seconds Minimum
Setting Time (at 37 ± 1°C)	4 minutes 0 seconds Minimum

1. Clean the preparation and restoration with oil-free prophypaste, water and air dry. Use calcium hydroxide if remaining dentin is thin and pulp protection is required.
2. Place 1 level scoop of powder and 2 bubble-free drops of liquid on glass block or mixing pad and mix thoroughly with flat bladed spatula to create a creamy consistency.
3. Spread a thin layer of cement over all inside surfaces of restoration and seat on preparation. Have patient bite down sufficiently hard on a cotton ball to remove air and trim excess cement squeezed out. The set cement is tooth coloured (U).

FEATURES

A radiopaque glass ionomer luting cement with prolonged working time followed by snap-set reaction. Combines excellent handling, low film thickness, high strength and contains fluoride. Excellent adhesion of porcelain, ceramic or metal to tooth structure.

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RECOMMENDED INDICATIONS

- ✓ Cementing of crowns, bridges, inlays or orthodontic brackets.

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- ✓ Pulpaüberkappung

STORAGE

- ✓ Store in a cool dry place (5-25°C). Close caps immediately after use..

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extracem®
Glass Ionomer Luting Cement
Glasionomer Befestigungszement

Zenit Cem

Self Ashesive Composite Luting Cement - Dual Cure

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Application

1. Remove the temporary restoration and carefully clean off any residual temporary luting cement from the cavity or prepared core. Rinse with a water spray. Dry using oil-free air, avoid over drying the dentine.
2. Verify the accuracy of the fit of the restoration.
3. Then clean the restoration thoroughly and pretreat as follows:

Etchable glass ceramic:

Etch the restoration with 5% hydrofluoric acid in accordance with the manufacturer's instructions, rinse thoroughly using a water spray and dry using oil-free air, apply silane (e.g. SilanePrimer) in accordance with the manufacturer's instructions.

Oxide ceramics, metal sand metal alloys:

Pre-treat in accordance with the manufacturer's instructions. Unless otherwise stated, sand blast the inside of the restoration with aluminumoxide (< 50 µm), clean with alcohol, and dry with air that is free of oil and water.

Fiber-reinforced root posts:

Clean the root post with ethanol, dry with oil-free air, and apply silane (e.g. SilanePrimer) in accordance with the manufacturer's instructions.

Indications

Permanent adhesive luting of indirect restorations (inlays, onlays, crowns, bridges, endodontic posts) made of noble metal, non-noble metal, PFM | Zirconia, glass ceramic (crowns and bridges) | Fiber reinforced composite (endodontic posts).

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Charecteristics

- ✓ Self adhesive
- ✓ High bond strength on enamel, dentin, metal and ceramics
- ✓ Radiopaque
- ✓ Dual-cure
- ✓ High mechanical properties

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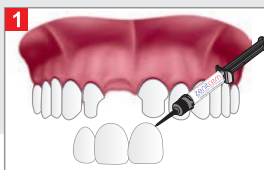
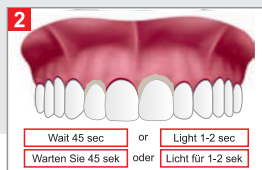
Apply **Zenit Cem** Self adhesive composite luting directly into the restoration using the tip. The working time of **Zenit Cem** is approx 1 minute. Place the restoration fully into position using slight pressure.

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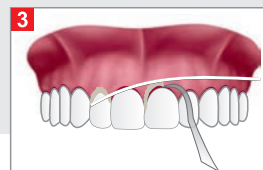
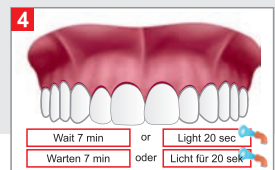
Allow the patient to slowly adopt the habitual occlusion. The restoration can support a full load after 7 minutes.

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Self-Curing
Excess material should be removed during the elastic phase, i.e. approximately 45 sec. to after application, using a single-use brush / brush / foam pellet/scaler or using dental floss. It is particularly important to ensure that excess is removed quickly in areas that are difficult to access (proximal, gingival margins).

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- 1x 25 g base
- 1x 25 g katalysator

DE

Temporary luting of inlays, onlays, crowns, partial crowns, bridges, attachments and posts. Temporary filling of small cavities.

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DE

Properties

- ✓ Excellent adhesion of temporary cementation
- ✓ Highly economical direct application
- ✓ No eugenol allergies
- ✓ Peelable consistency for easy clean-up

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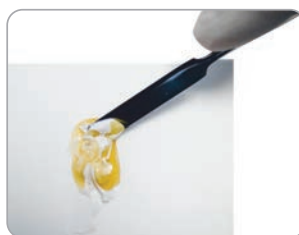
DE

Luting Cement

PROCEM
 Temporary eugenol-free luting cement

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- PRD.333.01.02 1x 2,5gr + Spitzen
- PRD.333.02.02 2x 2,5gr + Spitzen

DE

Novaseal is a light curing, radiopaque cavity liner with a very high pressure-strength and good adhesion to the dentin. It contains glassionomer, fluorides and could be applied directly, and could be set under all kinds of filling-materials.

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DE

Advantages

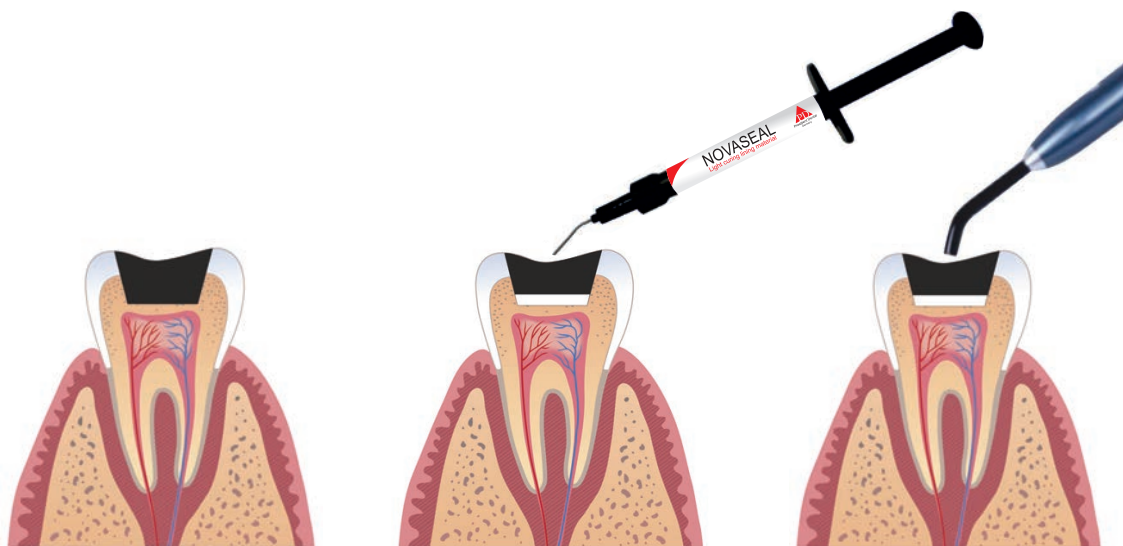
- ✓ Ready for use one component material
- ✓ **Time saving: Light curing in seconds**
- ✓ Quick and hygienic application
- ✓ High compressive strength (200 Mpa)
- ✓ Fluoride release against secondary caries
- ✓ High biocompatibility
- ✓ Radiopaque

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NOVASEAL
Light curing lining material

EndoPlus

Root Canal Sealer

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www.presidentdental.de

Product Code / Produktcode

PRD.100.01.290

Package Details

- 13.5 gr dual syringes (Base 9gr (Epoxy resin, Calcium phosphate) Catalyst 4.5g (Amines, Bismuth subcarbonate)
- Spatula
- Mixing Plate
- Dimension: 55x180x40 (mm)
- Weight: 13.5 gr

EN

Verpackungsdetails

- 13,5 gr Dual Spritze (Base 9gr (Epoxidharz, Calciumphosphat) Katalysator 4,5gr (Amine Bismut-Subcarbonat)
- Spatel
- Mischplatte
- Maße: 55x18x40 (mm)
- Gewicht: 13,5gr

DE

ENDOPLUS

Root Canal Sealer

EndoPlus Root Canal Sealing Material is a two component paste-paste dual syringe root canal sealer based on epoxy resin. It is the endodontic product for every dental practice and can be used for all established and more recent root canal sealing techniques

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DE

Properties

- ✓ Easy to mix by paste - paste
- ✓ Hermetic sealing ability
- ✓ Nonstaining to the teeth
- ✓ Insoluble in tissue fluids
- ✓ Working time of 35 minutes at 23°C (73°F)
- ✓ Setting time of 45 minutes at 37°C (99°F)
- ✓ Storage temperatures: +18°C (64°F) to +24°C (75°F)

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DE

Advantages

- ✓ Dual - syringes design provides for accurate dispensing
- ✓ Is homogeneous, free of air bubbles and easy to mix
- ✓ Provides a hermetic seal
- ✓ Penetrates superbly into the smallest lateral canals
- ✓ Does not stain teeth
- ✓ Has low solubility in soft tissues
- ✓ Has good apical tolerance
- ✓ Is radio - opaque

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DE

Gutta Percha Points

Gutta Percha is filler for root treatment and it is time saving

- ✓ Long shelf life
- ✓ Optimum working time
- ✓ Perfect Radiopasity

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Paper Points

Advantages

Paper Points are made of specially formulated absorbant paper and used for cleaning and absorbang purposes for root treatments

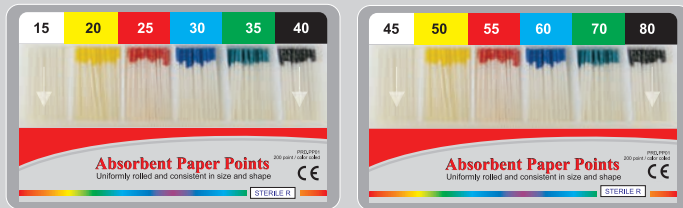
- ✓ It can be put through the canal without curving the paper
- ✓ It holds tight
- ✓ It is available in sterilized packages.

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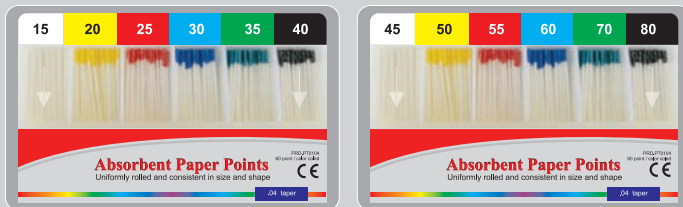
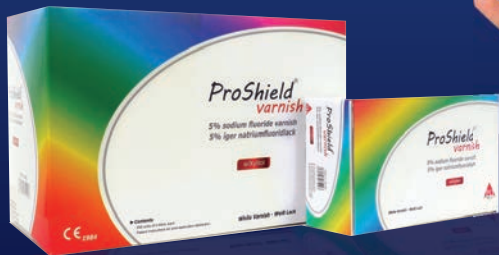
ProShield[®] varnish

5% sodium fluoride varnish

Proshield Varnish is fluoride varnish containing 50mg of sodium fluoride per 1ml. It also contains **TCP** and **Xylitol**. It is applied to teeth for the purpose of delivering fluoride and treatment hypersensitivity. It has four flavors such as bubblegum, strawberry, melon and mint. **Proshield Varnish** is packed in single dose package allows easy mixing and application.

Made
in
Germany
-best quality
-high standards

PRESIDENT DENTAL STUDY CLUB

"Follow President Dental Product Clinical Cases On
[President Dental Study Club](#)"

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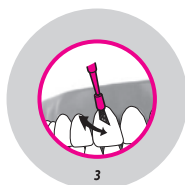
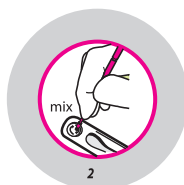
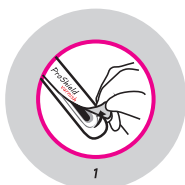
PRD.01.30100

Package Details

- Unit Dose - 0.4ml.
- Flavor: Bubblegum, Strawberry, Melon, Mint
- Contains
 - Standart pack : 32 single-unit-dose application sticks
 - Bulk pack : 200 single-unit-dose application sticks

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- ✓ Harz
- ✓ Natriumfluoride
- ✓ Ethanol
- ✓ Xylit
- ✓ TCP

Contra Indications

- ✓ Do not use on patients with ulcerative gingivitis or stomatitis.
- ✓ If patients are known to be allergic to ingredients including fluoride, resin or similar materials, **Proshield Varnish** should not be used.
- ✓ After the application of **Proshield Varnish** patients should avoid chewing hard food, drinking liquids containing alcohol and brushing for 4 to 6 hours.

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Indications

- ✓ For use as a desensitizing agent to help relieve dentinal hypersensitivity; To line cavity preparations under amalgam restorations
- ✓ Caries prophylaxis

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Produktcode / Verpackungsdetails

- PRD.01.4000 1x 300gr
- PRD.01.4015 20x 15gr

Prophy Powder is a spearmint flavored sodium bicarbonate powder for professional air polishing prophylaxis procedures. Used together with an air polishing system, **Prophy Powder** allows the removal of stains such as tobacco, coffee and tea. It is also suitable for Prophylaxis of orthodontic patients as well as preparation of tooth surfaces prior to bonding and sealant procedures.

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Clean

**PROPHY
POWDER**
CLEAN OF DIAMOND
SPEARMINT

Presigum

Hydrophilic Addition Silicone / A-Silicone

Additionsvernetzende Hyperhydrokompatible Silikone (Vinylpolysiloxane) / A- Silikon

www.presidentdental.de

Light Body / Fast Set

Product Code / Produktcode

PRD.01.PGUML01

Package Details / Verpackungsdetails

- 2x 50 ml
- 10x Mix. Tips
- 5x Yellow Oral Tips

Putty Kit Base + Catalyst Fast Set

Product Code / Produktcode

PRD.01.PGUMP01

Package Details / Verpackungsdetails

- 2x 250 ml

Light Body / Normal Set

Product Code / Produktcode

PRD.01.PGUML02

Package Details / Verpackungsdetails

- 2x 50 ml
- 10x Mix. Tips
- 5x Yellow Oral Tips

Putty Kit Base + Catalyst Normal Set

Product Code / Produktcode

PRD.01.PGUMP02

Package Details / Verpackungsdetails

- 2x 250 ml

Presigum Putty Soft: Vinylpolysiloxane for first impressions with elastic final consistency, recommended for all impression techniques: two-stage(double impression*) or single-stage (simultaneous).

Presigum Regular Body: Medium viscosity hydrophilic vinylpolysiloxane. Recommended for use in conjunction with Presigum Putty or Tray Material in the single-stage (simultaneous) technique, or as a monophasic material in a special tray.

Presigum Light Body: Low viscosity hydrophilic vinylpolysiloxane. Recommended for use in conjunction with Presigum Putty or Tray Material in the two-stage (double impression*) or single-stage (simultaneous) technique.

Presigum Super Light Body: Very low viscosity hydrophilic vinylpolysiloxane. Recommended for use in conjunction with Presigum Putty or Tray Material in the two-stage (double impression) or in the single-stage (simultaneous) technique.

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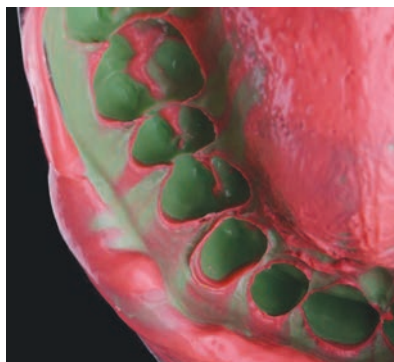
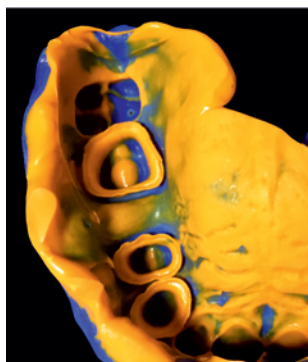
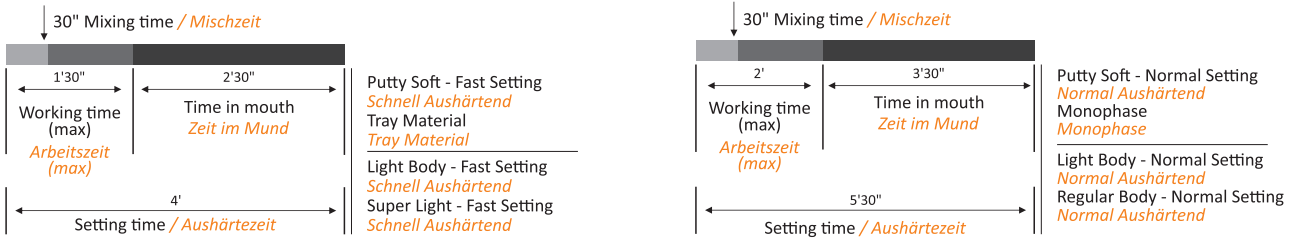
Presigum

Hydrophilic Addition Silicone_A-Silicone

Additionsvernetzende Hyperhydrokompatible Silikone (Vinylpolysiloxane)_A- Silikon

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Technical Data / Technische Daten

Presigum	Light Body Normal Setting	Light Body Fast Setting	Putty Soft Normal Setting	Putty Soft Fast Setting
ISO 4823	Type 3 Light-bodied Consistency	Type 3 Light-bodied Consistency	Type 0 Putty-bodied Consistency	Type 0 Putty-bodied Consistency
Mixing ratio Base: Catalyst <i>Dosierung Base: Katalysator</i>	1:1	1:1	1:1	1:1
Mixing Time <i>Anmischzeit</i>	in tube 30"	Auto mixing system	30"	30"
Total Working Time <i>Gesamtverarbeitungszeit</i>	2'	1'30"	2'	1'30"
Time in Mouth (min.) <i>Verweildauer im Mund (min.)</i>	3'30"	2'30"	3'30"	2'30"
Setting Time <i>Abbindezeit*</i>	5'30"	4'	5'30"	4'
Strain in Compression (min - max) <i>Verformung unter Druck (min - max)</i>	3-5%	3-5%	1-3%	1-3%
Recovery From Deformation <i>Rückstellung nach Verformung</i>	>99,5%	>99,5%	>99,0%	>99,0%
Linear Dimensional change 24 hours <i>Lineare Dimensionsabweichung nach 24 Stunden</i>	<0,20%	<0,20%	<0,20%	<0,20%
Hardness "Shore A" <i>Härte "Shore A"</i>	45±2	45±2	60±2	60±2

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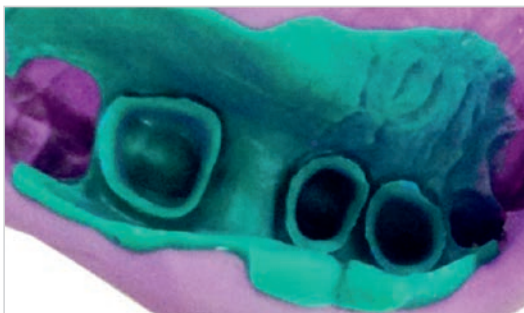
- 1600 gr / 900 ml

Physical Properties

• Color	: Purple
• Flavour	: All Fruits
• Mixing Time (sec.)	: ≤30
• Strain in Com.%	: 0.8/20
• Elastic Recovery (%)	: 96.5
• Detail Reproduction (µm)	: 75
• Compatibility With Gypsum (µm)	: 75
• Total Working Time (sec.)	: 60-90
• Setting Time (sec.)	: 340-420
• Hardness After 1h (sh-a)	: 67-77
• Linear Dimensional Change 24hrs (%)	: ≤1.5

EN

- ✓ Polysiloxane condensation curing preliminary impression material with high final hardness
- ✓ Simplified initial mixing through optimized fillers
- ✓ Silicate compound 63.30%
- ✓ Suitable for the double-mix technique
- ✓ Super detail reproduction
- ✓ Excellent characteristics
- ✓ Excellent dimensional accuracy and perfection in detail
- ✓ Easy to use and natural taste
- ✓ Hydrophilic surface characteristics resulting in low waste
- ✓ Hydroxy-polydimethylsiloxane 28.40%
- ✓ All fruits

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DE

Technical Details

Mixing Time	: 0:30 min
Processing Time	: 1:15 min
Time in Mouth	: 3:15 min
Setting Time	: 4:30 min
Strain in Compression	: 2-5%
Recovery From Deformation	: >98%
Linear Dimensional Change	: <0,2%

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Durosil S

C-Silicone Impression Material Type II
C-Silikon Abformmaterial Typ II

EN Low viscosity, extremely flowable, hydrocompatible polysiloxane
Ideal for double-mix technique, single-mix technique with two viscosities

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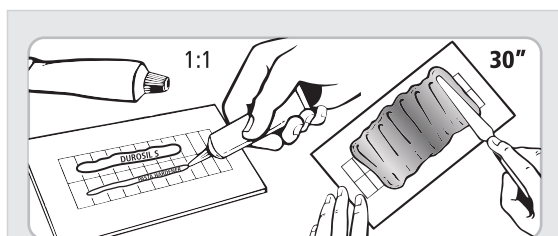
Physical Properties

• Color	: Green
• Flavour	: All Fruits
• Mixing Time (sec.)	: ≤30
• Strain in Com.%	: 2/20
• Elastic Recovery (%)	: 96.5
• Detail Reproduction (µm)	: 20
• Compatibility With Gypsum (µm)	: 50
• Total Working Time (sec.)	: 60-90
• Setting Time (sec.)	: 240-330
• Hardness After 1h (sh-a)	: 28-38
• Linear Dimensional Change 24hrs (%)	: ≤1.5

EN

- ✓ Very high compressive strength and recovery from deformation
- ✓ **DUROSIL S** consistency and high hydrocompatibility guarantee a perfect reproduction of fine details in every clinical condition
- ✓ Flowable consistency perfect flowability in the gingival sulcus
- ✓ Easy to use and natural taste
- ✓ Excellent dimensional accuracy and perfection in detail
- ✓ Silicate compound 48.90%
- ✓ Fruitmix 0.11%
- ✓ Hydroxy-polydimethylsiloxane 13.6%

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DE

Technical Details

Mixing Time	: 0:30 min
Processing Time	: 1:15 min
Time in Mouth	: 3:30 min
Setting Time	: 5:00 min
Strain in Compression	: 2-8%
Recovery From Deformation	: >99%
Linear Dimensional Change	: <0,7%

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EN For Durosil L, Durosil S and Paste Hardener for the condensation crosslinking Durosil Products Hardener is for silicone impression

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- 60ml

Technical Details

- ✓ Excellent flow characteristics
- ✓ No negative side effect
- ✓ Mucosal movements can be taken precisely
- ✓ Paste hardener for silicone
- ✓ Easy to mix due to outstanding wettability

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Physical Properties

- Color : Red
- Flavour : Mint
- Mixing Time (sec.) : ≤30
- Total Working Time (sec.) : 75-130
- Setting Time (sec.) : 75-130
- Consistency : 35-45

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Verpackungsdetails

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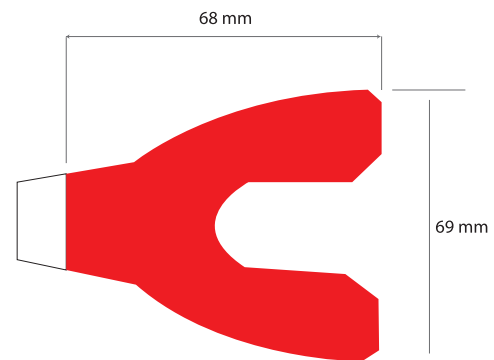
Technical Date

- ✓ It is used to identify the higher or lower chewing points on the tooth.
- ✓ Double sided, 80u thickness
- ✓ Easy, quick and precise marking
- ✓ No need to use tweezers
- ✓ Two dark colors (blue/red) are available for good marking

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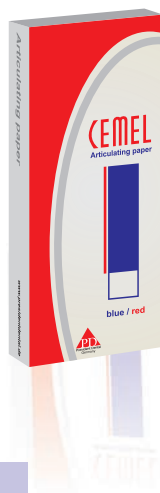
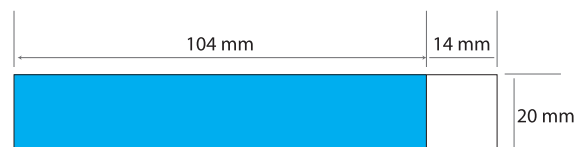
Presipack

www.presidentdental.de

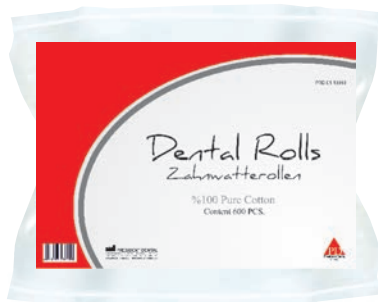
200 mt Sterilization rolls made of white medical paper coupled with a light blue polyester/polypropylene layer. The changing chemical color indicators verify the sterilization process by stream or ETO gas (ethylene oxide).

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Produktcode / Verpackungsdetails

- PRD.01.20020 1x 600 stk

- ✓ 100% highly absorbent cotton, pure white
- ✓ Flexibility, easy conformation, maintenance of the original shape when wet
- ✓ No cellulose or rayon fibers
- ✓ No metal, glass or grease
- ✓ Will not adhere to mucous membranes
- ✓ Well-packed for protection

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- ✓ Ear loop face mask (non-woven)
- ✓ 3-Ply protection, safe, comfort, highly breathable
- ✓ Fiberglass, latex free material, ideal for sensitive skin
- ✓ Adjustable nosepiece, soft and round ear loops
- ✓ Protection guarantee with over 99.8% bacterial filtration. Over 99.3% PFE, 160MM Hg, Delta P (H20/cm2)<5,0
- ✓ Fluid Resistant layer (meets all current ASTM 2100 performance criteria) with unique single pleated design to prevent fluid pooling reduce risk of contamination with fluid repellent outer and inner material

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Novamix

Alginate Mixer Alginatmischer

• The **Novamix** is the ultimate alginate mixer which is noise - free and capable of producing perfect mixtures with fine viscosity and no bubbles.

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DE

Features Include

- ✓ Streamlined and compactly designed outlook
- ✓ Cover body made of PVC for stronger pressure and heat resistance
- ✓ Long-lasting, enhanced motor delivering high-speed torsion and ensuring fine mixing
- ✓ Multiple memory settings allow to set different mixing time with only one button

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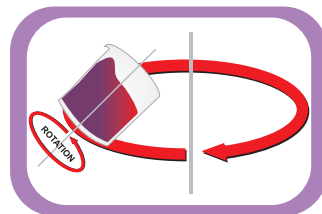
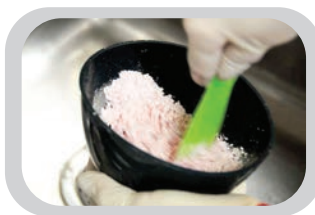
PRD.02.20002

Package Details

- 4 pcs mixing bowl - 200cc
- 1 pcs sized water container
- 1 pcs magnetic accessories container
- 1 pcs spatula

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- 1x 50 stck

Properties

- ✓ In comparison with other special tray materials, it is remarkably easy to use
- ✓ It has an indefinite working time
- ✓ It is non-sticky and easy to adapt
- ✓ It is easy to trim
- ✓ There is virtually no wastage - extremely cost - effective
- ✓ It can be light-cured with either UV or halogen light - there is no requirement for expensive dedicated light curing equipment
- ✓ It is extremely rigid and dimensionally stable - eliminating the risk of "hidden distortion" during the impression taking procedure and consequent errors, disputes and costs
- ✓ It is exceptionally strong, even in thin section
- ✓ No offensive smell

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Technical Data

- ✓ Spreadability : 14,0-19,0 mm
- ✓ Daylight Stability : mind. 20 Min.
- ✓ Flexural Strength : mind. 130 Mpa
- ✓ Curing of Both Sides : 3 - 5 Min.
(Depending on the device)
- ✓ Torsion Free

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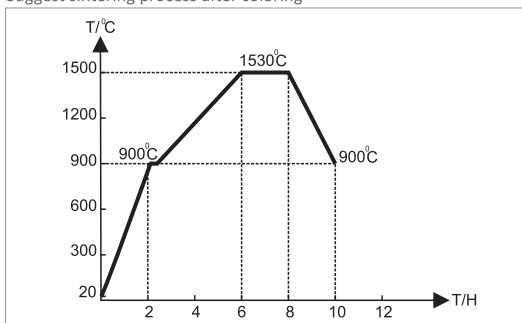
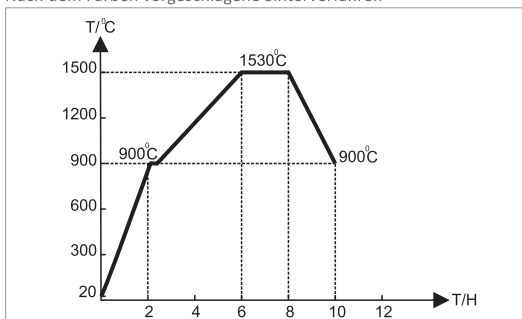
Finishing / Attached List

Suggest sintering process after coloring

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Zircon.X zirconia blocks are made by CIP and pre-sintering in low temperature. Although the product has rather strength, because of porosity, please handle carefully. When you receive the product, please check as below. **The general expansion coefficient (shrinkage) is around 1.2400 that is 19%**

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Chemical Composition and Powder Characteristic

Y ₂ O ₃	5.3 wt%
Al ₂ O ₃	0.25 wt%
SiO ₂	≤ 0.02 wt%
Fe ₂ O ₃	≤ 0.02 wt%
Na ₂ O	≤ 0.02 wt%
Aging Properties	Monoclinic Phase > 25%
Solubility	≤ 2000µg.cm ⁻²

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Coping and Bridges

Scanning and Designing ➡ Milling ➡ Cleaning ➡
 Dyeing ➡ Sintering ➡ Polishing ➡ Porcelain ➡
 Staining ➡ Finishing

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ZIRCON.X[®]

Advanced Ceramics for Crowns and Bridges

Super Translucency
 for full contour all restorations

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V: Veined

Preparation

Use the correct liquid with the powder (see chart). Shake well before use, especially if the shade is veined. Prepare the mould, carefully removing all traces of wax with boiling water with suitable detergent. Apply plaster separator.

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DE

Technical Data

- ✓ Powder : ProCryla Powder
- ✓ Liquid : ProCryla Liquid
- ✓ Mixing Ratio : Powder: 21 gr - Liquid:10 ml
- ✓ Dough Time : 20 - 30 Minuten
- ✓ Work Time : 15 - 20 Minuten
- ✓ Curing : Immerse in boiling water. Bring back to boil slowly. Boil for 20 minutes

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V: Veined

Packing and Pressing

Packing and Pressing The dough is at the right consistency when it is free from stickiness but not rubbery. The flask should be at room temperature when packed. Ensure the mould is properly filled as indicated by the excess dough being squeezed out around the edge. Close the flask slowly, a trial closure can be carried out if desired. Never pack the dough after the work time has elapsed or the material has become rubbery, it is advisable to keep the flask under pressure for 15 minutes before curing.

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V: Veined

Mix Cold Cure Powder and Liquid to form a selfcure (autopolymerising) acrylic resin. Conforms to BS EN ISO 20795-1 Type II Class I For repairs, indirect relines and additions to removable prosthetic appliances. Will bond to acrylic teeth when cured by hydroflask.

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Technical Data

- ✓ Powder : ProCryla Cold Cure Powder
- ✓ Liquid : ProCryla Cold Cure Liquid
- ✓ Mixing Ratio : Powder: 21 gr - Liquid:10 ml
- ✓ Dough Time : 20 - 30 Minuten
- ✓ Work Time : 15 - 20 Minuten
- ✓ Curing : In hydroflask at 40°C for 10-20 minutes at 30psi Remove carefully, trim and polish. Will also self cure in 20 minutes.

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Risk Phrases

Highly Flammable Flash point below 11°C Harmful by inhalation, contact with skin and if swallowed. Danger of cumulative effects. Irritating to eyes, respiratory system and skin. May cause sensitisation contact.

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Safety Phrases

Keep container in well ventilated place. Keep away from sources of ignition-no smoking. Do not empty in to drains. Take precautionary measures against static discharge. Avoid prolonged exposure to sunlight. in case of contact with eyes and skin: Flush with copius volumes of water. Seek medical attention. Store below 25°C. Replace cap immediately after use

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Cobalt based alloys (non-precious) are used in the dental industry for making dental prosthesis. They are economical alternative to Gold & Silver based alloys (precious) but perform well due to their mechanical properties and biocompatibility. **President Dental** manufactures a range of Non Precious Dental alloys using the continuous cast method. We have all the relevant certificates such as 93/42/EEC&ISO 13485:2003 in order to manufacture dental base metal casting alloys.

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Properties

- ✓ High corrosion resistant
- ✓ Bio-compatibility
- ✓ Easy processing and shaping due to its excellent elasticity
- ✓ Superior mechanical characteristics with regard to precious metal alloy
- ✓ Melting in any kind of induction and centrifuge machine
- ✓ Easy fluidity control and hygiene
- ✓ Low thermal conductivity
- ✓ High purity, due to production is made in continuous casting process
- ✓ Permanent metallurgical quality control with modern controlling processes
- ✓ Pure alloys

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Nickel based alloys (non-precious) are used in the dental industry for making dental prosthesis. They are economical alternative to Gold & Silver based alloys (precious) but perform well due to their mechanical properties and biocompatibility. **President Dental** manufactures a range of Non Precious Dental alloys using the continuous cast method. We have all the relevant certificates such as 93/42/EEC & ISO 13485:2003 in order to manufacture dental base metal casting alloys.

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Properties

- ✓ Excellent adaptation with ceramic powders
- ✓ Binds up elasticity
- ✓ Bio-compatibility
- ✓ Contains no beryllium
- ✓ Easy shaping
- ✓ Melting in any kind of induction and centrifuge machine
- ✓ Easy fluidity control and hygiene
- ✓ Low thermal conductivity
- ✓ Pure alloys

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Synthetic Abrasive

White and Brown Synthetic Abrasive

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The above features enable the product to be adequately practical for the production of all kinds of bonded abrasives: surface grinding, external and cylindrical grinding, profile grinding, creepfeed grinding of low or unalloyed steel up to 62 HRC hardness; gear grinding and thread grinding.

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Properties

- ✓ Sharp-Edged grains
- ✓ Superior hardness
- ✓ High loadbearing capacity
- ✓ Excellent chemical stability

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- 1x 500 gr

Properties

- ✓ High strength and rigidity at mouth temperature
- ✓ Wide softening range above mouth temperature
- ✓ Easily mouldable in the softened state, without flaking, cracking or tearing
- ✓ There should be low thermal contraction
- ✓ Easily carvable at room temperature without flaking or chipping
- ✓ There should be little change in properties on melting and re-solidification
- ✓ There should be no residue on boiling out

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Crown Bridge Investment

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www.presidentdental.de

Ceravest is a carbon-free phosphate investment developed for professionals who demand a more finely-grained investment. It is designed for use with all classes of casting alloys: precious, semiprecious, and non-precious.

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DE

Properties

- ✓ Extremely fine texture results in accurate castings with very smooth surfaces
- ✓ Extended working time, as much as ten minutes, allows the investing of multiple flasks
- ✓ Absence of carbon assures compatibility with all casting alloys, especially palladium and non-precious

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DE

Technical Data

- | | |
|---|--------------------------------------|
| ✓ Liquid*/Powder Ratio : 22.6ml./100 gr | ✓ Initial Setting Time : 5 min |
| ✓ Working Time : 4-5 minutes | ✓ Thermal Expansion : 1,33% |
| ✓ Vacuum Mixing : 60 Sec | ✓ Compressive Strength, Wet: 600 psi |
| ✓ Hand Mixing in a bowl: 60 Sec | ✓ Special Liquid : 15% |
| ✓ Fluidity : 170-180 mm | ✓ E-Module : 75% |

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