



Universal Hybrid Composite Restorative Option 1 Composite Restorative Restorative





- ✓ Easy handling, non-sticky formula that won't slump.
- ✓ Excellent wear resistance and superior performance at the margin.
- ✓ Resists chipping and fracturing for long-lasting restorations.
- Creamy consistency
- Great optical properties
- Simple technique
- Excellent polishability
- Long working time
- •Suitable for all cavities in Anterior & Posterior teeth

DYNAMIC PLUS Range is a complete range of restorative materials that offer:-


- Easy handling
- Versatility
- Clinical performance

The result is the ideal combination of performance and value.

DYNAMIC PLUS is a visible light cure microhybrid composite for anterior and posterior restorations. DYNAMIC PLUS is based on BIS-GMA-resin and inorganic filler particles of 0.05-1 cm. The total filler load is 80%, and the total filler volume is 65%.

It is available as single syringes or as kits in various combinations containing a bonding system and application accessories and various number of syringes. (5 syringe Starter Kit and 8 syringe Regular Kit). Shades: A1, A2, A3, A3, 5, A4, B1, B2, B3, C2, C3, D3, OA2, OA3, I, P, E


Compressive Strength (MPa) Load applied - P 200 150 Dynamic Gradia Spectrum Versacomp A. TPH

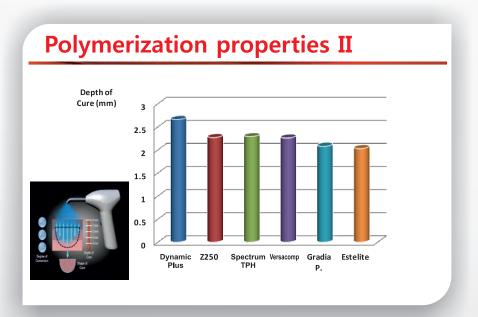


Polymerization properties Polymerization Shrinkag (vol%) 3.5 2.5 2 1.5 Rigid slide Light source Polymerization properties Dynamic Versacomp Gradia Spectrum Z250 ClearFil TPH

Compressive strength


The ability of a material to resist fracture / breakage from compressive (or crushing) forces from a single impact.

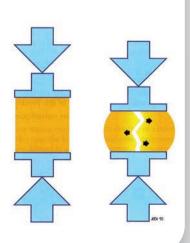

Clinical Steps of Application 1) Enamel and dentine pre-treatment Start at enamel Extend to dentine Rinse with Water then dry with air rinse 15 sec Do not desiccate

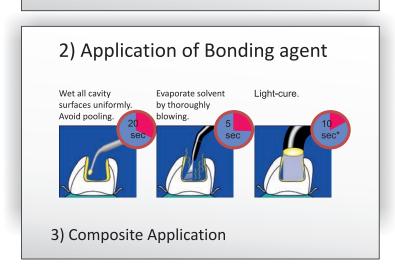



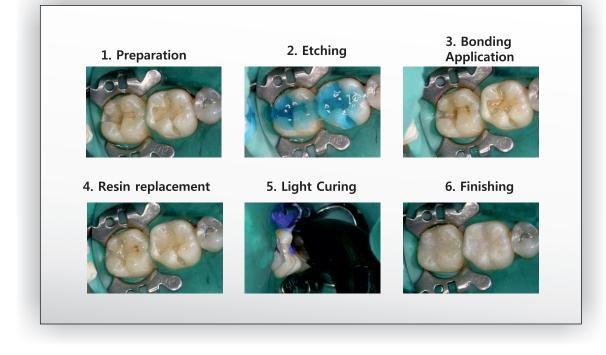

LAYER TECHNIQUE

ANTERIOR





PREPARATION OF THE TOOTH





	Shade	A1	A2	A3	A3.5	A4
1000	Enamel	A1	A2	А3	A3.5	Α4
	Dentine	A2 2 mm	A2 2.0 mm	A3 2.0 mm	A3 2.0 mm	A4 Enamel
NOOD RA	Enamel	A1 0.5 mm	A2 0.5 mm	A3 0.3 mm	A3.5 0.5 mm	A4 0.2 mm





PRD.100.01.100	5x4 gr syringe	1x5 ml.Prebond	1x3 ml. syringe etching gel
Starter Kit	(A1,A2,A3,B1,B2)	(Bonding Agent)	
PRD.100.0.101	8x4 gr syringe	1x5ml. Prebond	2x3 ml. syringe etching gel
Regular Kit	(A1,A2,A3,A3-5,B2,B3,A2Q,I)	(Bonding Agent)	

LAYER TECHNIQUE

POSTERIOR






PREPARATION OF THE TOOTH





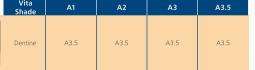





















A1	PRD.100.01.110	C2
A2	PRD.100.01.111	C3
A3	PRD.100.01.112	D3
A3,5	PRD.100.01.113	OA2
A4	PRD.100.01.114	OA3
B1	PRD.100.01.115	I
B2	PRD.100.01.116	P
В3	PRD.100.01.117	Е
	A2 A3 A3,5 A4 B1 B2	A2 PRD.100.01.111 A3 PRD.100.01.112 A3,5 PRD.100.01.113 A4 PRD.100.01.114 B1 PRD.100.01.115 B2 PRD.100.01.116